

# Dermapen 4™ PATIENT PRE-TREATMENT FORM

With over three million treatments performed worldwide by **Authorised Treatment Providers**, DermapenWorld is the innovator of automated microneedling – **original, trusted, Australian**.

Dermapen 4™ has won multiple awards and has become one of the most trusted names in the medical aesthetics industry. Thousands of practitioners endorse Dermapen 4's world-leading technology, which helps 'rewrite skin history' without side effects or downtime.

Dermapen 4's oscillating action effortlessly glides over the skin to stimulate the body's natural healing and regenerative response. When combined with Dp Dermaceuticals Meso-Glides and ÜBER PEEL, the treatment becomes even more versatile and can target the appearance of:

- Ageing
- Wrinkles
- Uneven skin tone
- Uneven skin texture
- Stretch marks
- Scarring
- Alopecia

Dermapen 4™ may be safely used for face, body, scalp and intimate rejuvenation.

Dermapen 4™ treatments are fast, effective, comfortable and offer visual results after just one session. Based on your areas of concern, your DermapenWorld **Authorised Treatment Provider** will recommend a series of treatment in conjunction with Dp Dermaceuticals home care products for optimal results and maintenance.

If you are experiencing any of the following conditions or concerns, on or around the treatment area, you will need to let your practitioner know prior to treatment:

- Active Psoriasis
- Active Herpes simplex (cold sores)
- Scleroderma
- Open lesions
- Skin cancer
- Undiagnosed/unusual moles
- Active lupus erythematosus
- Active collagen vascular disease
- Acne vulgaris stage IV
- Warts
- Bacterial/fungal infections
- Undiagnosed solar keratosis
- Haemophilia (or other blood clotting disorders)
- Active vitiligo
- History of keloid scars

Also, inform your practitioner if you are (or planning to get) pregnant, have any cosmetic tattooing, have recently experienced any chronic illness or are undergoing major medical care.

If you are doubt regarding any health condition or medications that you may take, always seek medical advice from your physician.

Certain health conditions, medications, supplements and lifestyle factors may affect a Dermapen 4™ procedure. All patients are required to complete a DermapenWorld Consultation Form prior to treatment for assessment by an **Authorised Treatment Provider**.

All patients should prepare and prime their skin for at least 2 weeks prior to treatment with Dp Dermaceuticals. Your practitioner will prescribe a customised skin solution for pre-op, post-op and ongoing maintenance.

## COMFORT

Your Dermapen 4™ **Authorised Treatment Provider** will take all steps to ensure total comfort during for your Dermapen 4™ procedure. If at any point you feel discomfort, please inform your practitioner immediately.

## PROTECTION & SAFETY

Your Dermapen 4™ treatment only uses sterile, single-use consumables throughout the procedure, ensuring complete health and safety.

## TREATMENT DURATION

Please allow a minimum of 20-30 minutes for your Dermapen 4™ clinical treatment including preparation, combination therapies and post-care.



# Dermapen 4™ PATIENT POST-OPERATIVE FORM

It is important to protect and provide extra special care for your skin for at least 3-4 days following a Dermapen 4™ procedure. Typically, patients experience 36-48 hours of mild to moderate sunburn in appearance and sensation following their treatment.

During the healing process, minor itching, hives, some flaking, or redness may appear. If symptoms persist, please call your Dermapen™ Authorised Treatment Provider.

Do not pick, squeeze or agitate the skin during the recovery period.

Please avoid the following activities for up to 2 days following a Dermapen 4™ treatment:

- Direct ultra-violet exposure (sun and solariums/sun beds)
- Intensive cardio, exercise or gymnasium regimens
- Excessively hot showers, bathing, spas or sauna
- Further aesthetic treatments (including, but not limited to, microdermabrasion, laser, intense pulsed light, chemical peels, muscle relaxant injections and dermal fillers)
- Spray or self-tanning
- Swimming in chlorinated pools or the ocean
- Tattooing (including cosmetic tattooing)

It is recommended to only use Dp Dermaceuticals protocol products for post-operative care. Your practitioner will most likely suggest:

- **Cleanse:** TRI-PHASE CLEANSER™
- **Serum:** HYLA ACTIVE™
- **Correct/Moisturise:** VITAMIN RICH REPAIR™
- **Protect:** COVER RECOVER™

Dp Dermaceuticals COVER RECOVER™ delivers a broad spectrum SPF30/PA+++ and camouflage in one. It should be re-applied 2-3 times a day.

Depending on your skin concern and response, your practitioner may also prescribe other targeted products, including CLINIPREP™ and SSSCAR™.

Please avoid the use of skin care products containing any of the following active resurfacing ingredients for up to 5 days following a Dermapen 4™ treatment:

- Mechanical exfoliants (scrubs)
- Alpha hydroxy acids (AHAs) (including, but not limited to, glycolic, lactic or malic acid)
- Beta hydroxy acid (BHA) including salicylic acid
- Benzoyl peroxide
- Retinoids (including, but not limited to, tretinoin, adapalene, tazarotene and retinol)
- Hydroquinone
- Bleaching agents
- Alcohol (including, but not limited to, isopropyl alcohol/de-natured alcohol/rubbing alcohol)

Your regular Dp Dermaceuticals regimen may be altered by your practitioner, according to your individual needs and skin condition.

If in doubt with any of the above activities or products, please call your Authorised Treatment Provider for clarification. For any isolated, adverse reactions, seek medical assistance.

